



Disposable MediQuilt® Cover

- Single patient use reduces cross contamination risk
- Made from durable and breathable material for superior patient comfort
- Unique design for quick and easy removal of the Quilt
- Cover opening can be sealed to retain the quilt and prevents fluid contact
- Meets BS 71756:1959, source 5 fire standards



Instructions:

To cover MediQuilt®

- Remove MediQuilt® and MediQuilt® cover from packaging
- Insert the two shorter corners of MediQuilt® inside the two far corners of the MediQuilt® cover
- Grasp the two corners and shake to ensure MediQuilt® is covered completely
- To seal the opening lay the quilt on the stretcher and pull away the double sided tape at the bottom of the cover and press the seams together

Specifications:

Product code :	BL210-PK
Material :	40 grm/m2 Polypropylene
Colour :	Blue
Dimensions :	173 cm x 111 cm
Pack Size :	Pk 25

www.medilinen.com

info@medilinen.com

